Automatic Upper Arm Blood Pressure Monitor

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Intellisense

All for Healthcare

Read Instruction manual ① and ② before use.

- FR** Lire le mode d'emploi ① et ② avant l'utilisation.
- DE** Lesen Sie vor der Verwendung Gebrauchsanweisung ① und ②.
- Italiano** Leggere il manuale di istruzioni ① e ② prima dell'uso.
- Español** Lea el manual de instrucciones ① y ② antes del uso.
- Nederlands** Lees gebruiksaanwijzing ① en ② voor gebruik.
- Русский** Перед использованием прочтите руководства по эксплуатации ① и ②.
- Türkçe** Kullanmadan önce kullanım kılavuzu ① ve ②'yi okuyun.

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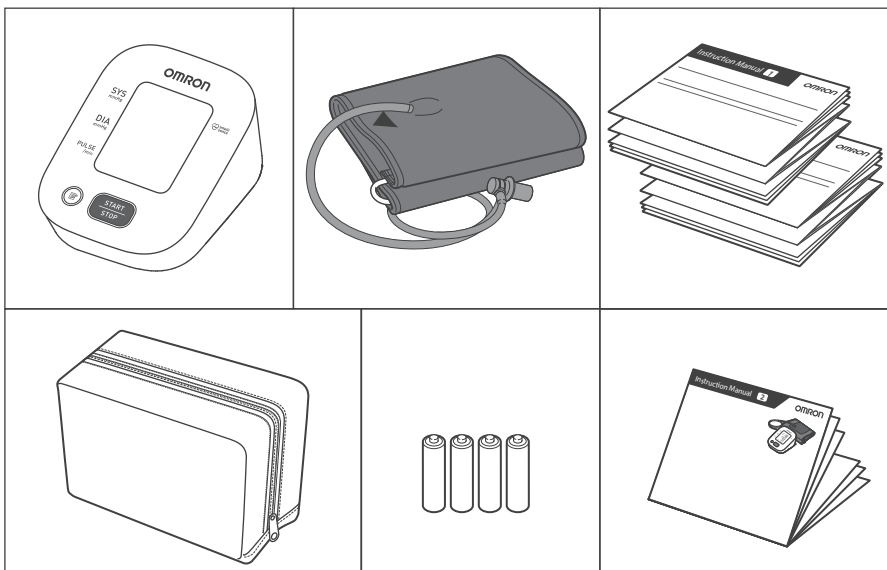
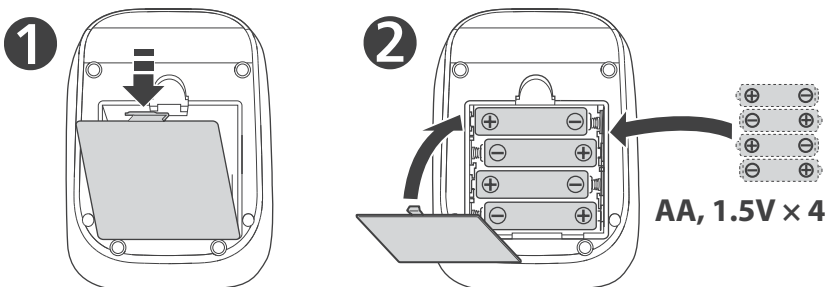
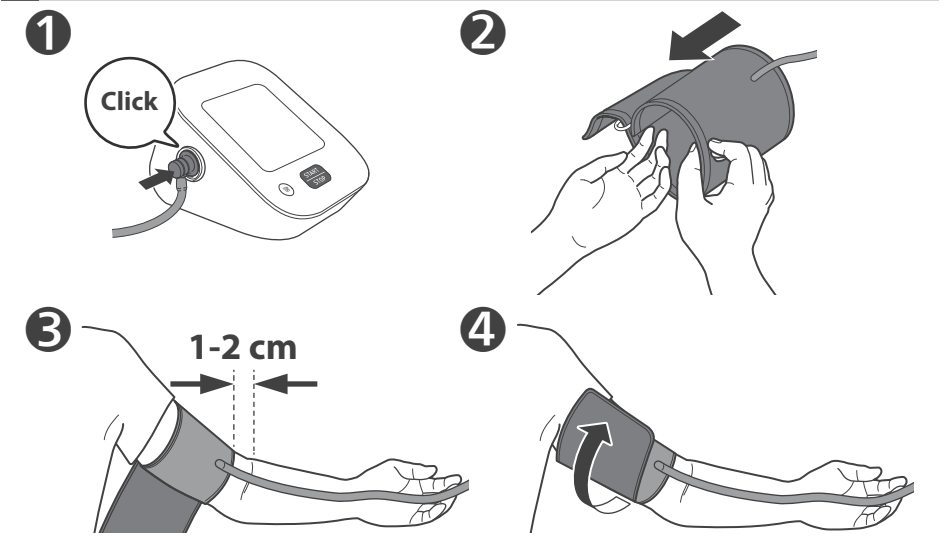
3 Tube side of the cuff should be 1 - 2 cm above the inside elbow.

- FR** Le côté tuyau du brassard doit être positionné 1 à 2 cm au-dessus de l'intérieur du coude.
- DE** Das Manschettenstück mit dem Schlauch muss 1 bis 2 cm oberhalb des Ellbogens liegen.
- IT** Il lato del bracciale con il tubo deve trovarsi al di sopra del gomito interno, a una distanza di circa 1 o 2 cm.
- ES** El lado del tubo del manguito deberá quedar 1 o 2 cm por encima de la parte interna del codo.
- NL** De kant met de slang van de manchet moet 1 - 2 cm boven de binnenkant van de elleboog liggen.
- RU** Край манжеты с трубкой должен находиться на 1–2 см выше локтевого сгиба.
- TR** Kolluğun boru tarafı, dirsek içinin 1 ila 2 cm üstünde olmalıdır.

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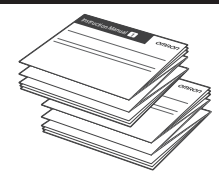
4 Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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If taking measurements on the right arm, refer to:

- FR** Pour la prise de mesures au bras droit, voir :
- DE** Bei Messungen am rechten Arm siehe:
- IT** Se la misurazione viene eseguita al braccio destro, fare riferimento a:
- ES** Si va a realizar mediciones en el brazo derecho, consulte:
- NL** Als u metingen aan de rechterarm uitvoert, raadpleeg dan:
- RU** При измерении на правой руке см.
- TR** Sağ koldan ölçüm yapıyorsanız aşağıdakilere bakın:

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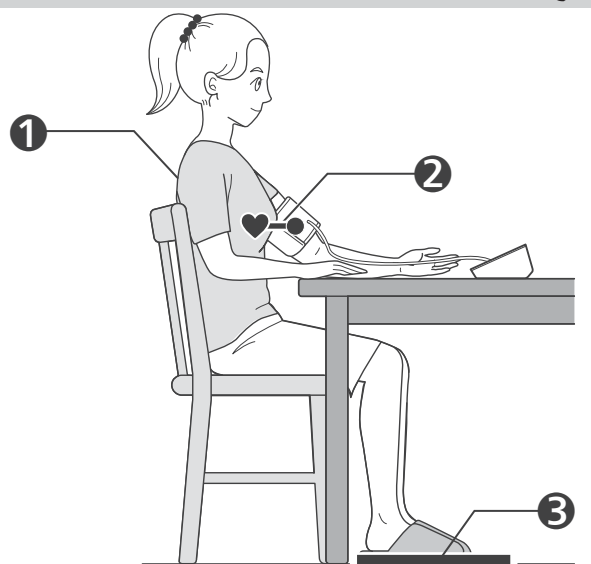
1 Sit comfortably with your back and arm supported.

- FR** S'asseoir confortablement de manière à ce que le dos et le bras soient bien soutenus.
- DE** Bequem hinsetzen, mit dem Rücken anlehnen und den Arm auf eine Unterlage legen.
- IT** Sedere comodamente con la schiena e il braccio ben sostenuti.
- ES** Siéntense cómodamente de modo que la espalda y el brazo estén bien apoyados.
- NL** Neem een comfortabele zitpositie in, met uw rug en arm ondersteund.
- RU** Сядьте удобно, чтобы спина и рука опирались на что-либо.
- TR** Sirtınız ve kolunuz desteklenerek rahat bir şekilde oturun.

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2 Place the arm cuff at the same level as your heart.

- FR** Le brassard doit se trouver au même niveau que votre cœur.
- DE** Die Manschette auf Herzhöhe platzieren.
- IT** Posizionare il bracciale allo stesso livello del cuore.
- ES** Coloque el manguito al mismo nivel que el corazón.
- NL** Plaats de armmanchet op hetzelfde niveau als uw hart.
- RU** Манжета должна находиться на уровне сердца.
- TR** Kolluğu kalbinizle aynı düzeye getirin.

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3 Keep feet flat, legs uncrossed, remain still and do not talk.

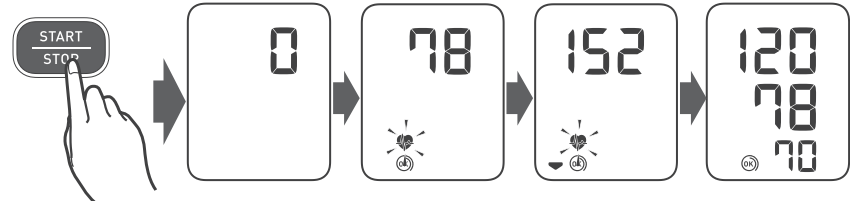
- FR** Garder les pieds à plat, les jambes non croisées, ne pas bouger et ne pas parler.
- DE** Füße flach auf den Boden stellen, die Beine nicht kreuzen, still halten und nicht sprechen.
- IT** Tenere i piedi ben poggiati, non incrociare le gambe, rimanere fermi e non parlare.
- ES** Mantenga los pies planos, las piernas sin cruzar, quédese quieto y no hable.
- NL** Houd de voeten plat, benen niet gekruist, zit stil en praat niet.
- RU** Поставьте ноги ровно, не перекрещивая их, не двигайтесь и не разговаривайте.
- TR** Ayaklarınızı düz, bacaklarınızı açık tutun, hareketsiz kalın ve konuşmayın.

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When the [START/STOP] button is pressed, the measurement is taken and saved automatically.

- FR** Après une pression sur le bouton [START/STOP], la mesure démarre et le résultat est enregistré automatiquement.
- DE** Beim Drücken der Taste [START/STOP] erfolgt die Messung. Die Messwerte werden automatisch gespeichert.
- IT** Quando si preme il pulsante [START/STOP], la misurazione viene eseguita e salvata automaticamente.
- ES** Al pulsar el botón [START/STOP], se obtiene la medición, que se guarda automáticamente.
- NL** Wanneer op de knop [START/STOP] wordt gedrukt, wordt de meting automatisch uitgevoerd en opgeslagen.
- RU** После того, как кнопка [START/STOP] нажата, измерение выполняется и сохраняется автоматически.
- TR** [START/STOP] düğmesine basıldığında ölçüm yapılır ve otomatik olarak kaydedilir.

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If your systolic pressure is more than 210 mmHg: After the arm cuff starts to inflate, press and hold the [START/STOP] button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure.

- FR** Si votre pression systolique est supérieure à 210 mmHg :
- DE** Wenn Ihr systolischer Druck höher ist als 210 mmHg:
- IT** Se la pressione sistolica è superiore a 210 mmHg:
- ES** En caso de que su presión arterial sistólica esté por encima de 210 mmHg:
- NL** Als uw systolische druk hoger is dan 210 mmHg:
- RU** Если систолическое давление выше 210 мм.рт.ст.:
- TR** Sistolik basıncınız 210 mmHg'den fazlaysa:

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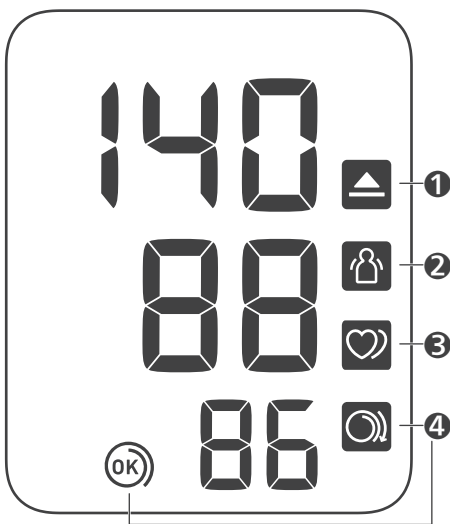
1 Appears if "SYS" is 135 mmHg or above and/or "DIA" is 85 mmHg* or above.

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2 Appears when your body moves during a measurement. Remove the arm cuff, wait 2 - 3 minutes and try again.

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3 Appears when an irregular rhythm** is detected during a measurement. If it appears repeatedly, OMRON recommends to consult your physician.

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OMRON SANTÉ FRANCE SAS www.omron-healthcare.com/distributors